Dilution Instructions for Wescor Concentrated Hematology Reagents for use with the Aerospray® Model 7150

English

Required Supplies and/or Equipment:

AC-043: 500 mL bottle
Graduated cylinders
Filtered Deionized Water
Anhydrous, Reagent Grade Methanol
AC-072: 5 L bottle

Optional Supplies and/or Equipment:

Anhydrous, Reagent Grade Ethanol or Reagent Alcohol

If bottles are reused be sure to clean the bottle thoroughly. Use 1/10 mixture of household bleach to disinfect the bottles.

WARNING! These reagents contain moderately toxic chemicals that require care in handling. Always use appropriate safety measures including gloves and eye protection when handling reagents. Refer to material safety data sheet (MSDS).

SS-171A Hematology Reagent A, Buffer (pH 6.8) Concentrate (30 mL);
SS-171A500 Hematology Reagent A, Buffer (pH 6.8) Concentrate (500 mL);
SS-172A Hematology Reagent A, Buffer (pH 7.2) Concentrate (30 mL);
SS-172A500 Hematology Reagent A, Buffer (pH 7.2) Concentrate (500 mL);

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-071A or SS-072A):

1. Obtain an empty 5 L bottle.
2. Add 30 mL of SS-171A, SS-171A500, SS-172A, or SS-172A500 to 5 L deionized water.
3. Mix well.
4. Label new bottle with supplied label.
5. Write LOT number and expiration date from concentrate bottle on new label.

SS-171B2 Hematology Reagent B, Thiazin Stain Concentrate (200 mL)

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-071B):

1. Add 300 mL Methanol to SS-171B2 bottle.
2. Mix well.
3. Check box on label when mixed.

SS-171C2 Hematology Reagent C, Eosin Stain Concentrate (220 mL)

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-071B):

1. Add 280 mL Methanol to SS-171C2 bottle.
2. Mix well.
3. Check box on label when mixed.

SS-148 Aerofix® Additive for Methanol

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-048):

1. Obtain an empty 500 mL bottle.
2. Add 15 mL SS-148 to 500 mL Methanol
3. Mix well.
4. Label new bottle with supplied label.
5. Write LOT number and expiration date from concentrate bottle on new label.

SS-029C Nozzle Cleaning Solution Concentrate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-029):

1. Add 250 mL Methanol or Ethanol to bottle of SS-029C.
2. Mix well.
3. Check box on label when mixed.

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