## Wescor AFB Reagent Concentrate Dilution Instructions English

Required Supplies and/or Equipment:  
AC-043 – empty 500 mL bottle  
Label supplied with bottle of concentrate  
Graduated cylinders  
Filtered Deionized Water  
Anhydrous, Reagent Grade Isopropanol  
Anhydrous, Reagent Grade Methanol  
Anhydrous, Reagent Grade Acetone  
Anhydrous, Reagent Grade Ethanol or Reagent Alcohol  
Optional Supplies and/or Equipment  
AC-038 – 5 L space-saver container

If bottles are reused be sure to clean the bottle thoroughly. Use 1/10 mixture of household bleach to disinfect the bottles.

**WARNING!** These reagents contain moderately toxic chemicals that require care in handling. Always use appropriate safety measures including gloves and eye protection when handling reagents. Refer to material safety data sheet (MSDS).

### SS-161A AFB Reagent A, Carbol Fuchsin Decolorizer Concentrate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-061A):

1. Obtain an empty 500 mL bottle.
2. Add 30 mL SS-161A to 470 mL Methanol
3. Mix well.
4. Label new bottle with supplied label.
5. Write LOT number and expiration date from concentrate bottle on new label.

### SS-161AF AFB Reagent A, Fluorescence Decolorizer Concentrate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-061AF):

1. Obtain an empty 500 mL bottle.
2. Add 30 mL SS-161A to 350 mL Isopropanol and 120 mL Deionized Water.
3. Mix well.
4. Label new bottle with supplied label.
5. Write LOT number and expiration date from concentrate bottle on new label.

### SS-161BBG AFB Reagent B, BBG Brilliant Green Concentrate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-061BBG):

1. Obtain an empty 500 mL bottle.
2. Add 32.5 mL SS-161BBG to 417.5 mL Deionized Water and 50 mL Ethanol.
3. Mix well.
- 4.
5. Write LOT number and expiration date from concentrate bottle on new label.

### SS-161BMB AFB Reagent B, Methylene Blue Concentrate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-061BMB):

1. Obtain an empty 500 mL bottle.
2. Add 32.5 mL SS-161BMB to 417.5 mL Deionized Water and 50 mL Ethanol.
3. Mix well.
4. Label new bottle with supplied label.
5. Write LOT number and expiration date from concentrate bottle on new label.

### SS-161BP AFB Reagent B, Potassium Permanganate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-061BP):

1. Obtain an empty 500 mL bottle.
2. Add 50 mL SS-161BP to 450 mL Deionized Water.
3. Mix well.
4. Label new bottle with supplied label.
5. Write LOT number and expiration date from concentrate bottle on new label.

### SS-161M AFB Mycophol Cell Adhesive Concentrate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-061M):

1. Obtain an empty 500 mL bottle.
2. Add 50 mL SS-161M to 450 mL normal saline.
3. Mix well.
4. Label new bottle with supplied label.
5. Write LOT number and expiration date from concentrate bottle on new label.

### SS-029C Nozzle Cleaning Solution Concentrate

Mixing Instructions (when mixed according to these instructions the product is equivalent to SS-029):

1. Add 250 mL Methanol or Ethanol to bottle of SS-029C.
2. Mix well.
3. Check box on label when mixed.

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### SS-161A AFB Reagent A, Carbol Fuchsin Decolorizer Concentrate.

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