

**ABBREVIATED INSTRUCTIONS
MACRODUCT®
SWEAT COLLECTION SYSTEM**

CAUTION!

THESE INSTRUCTIONS ARE INTENDED FOR REFERENCE ONLY. DO NOT ATTEMPT TO USE THIS SYSTEM FOR SWEAT INDUCTION AND COLLECTION UNTIL YOU HAVE READ AND THOROUGHLY UNDERSTAND THE COMPLETE PROCEDURES DETAILED IN THE INSTRUCTION MANUAL.

1. Assemble equipment and supplies.
2. Inspect electrodes and connect to inducer.
3. Clean selected skin areas.
4. Install PIILOGEL® Discs on both electrodes.
5. Attach electrodes to limb.
6. Activate iontophoresis.
7. Prepare MACRODUCT sweat collector during iontophoresis.
8. Remove electrodes at completion of iontophoresis.
9. Clean skin under positive (red) electrode.
10. Attach MACRODUCT sweat collector firmly to limb.
11. Remove and store sweat sample.
12. Remove and discard MACRODUCT residue.
13. Clean electrodes.

FOR PROMPT SERVICE:
Phone (435) 752-6011
Toll Free (800) 453-2725
Fax (435) 752-4127
Email sales@wescor.com

L181A